



Dedeco International, Inc.
 11617 State Route 97, Long Eddy, N.Y. 12760-0244
 phone: (845) 887-4840 • fax: (845) 887-5281
 email: service@dedeco.com • web: www.dedeco.com

Sterilization Procedures

- Prior to first use, instruments should be stored in original packaging at room temperature and protected against dust and humidity.
- Instruments should be disinfected, cleaned and sterilized before and after each use.
- Use cleaning agents with anti-corrosive properties. Tungsten carbide and stainless steel instruments may corrode in cold sterilization, resulting in discolorization and shorter service life.
- Hydrogen peroxide should not come in contact with tungsten carbide as this will damage and reduce the service life of the instrument.
- Avoid temperatures above 356°F. Excessive heat will lead to material fatigue and reduce service life.
- After cleaning, rinse instruments thoroughly with distilled water and dry immediately.
- Contact between instruments during cleaning and disinfection procedures may cause damage and cross contamination.

Instruments	Type of Sterilization				
	Dry Heat Oven	Steam Autoclave	Chemical Vapor	EIO (Ethylene Oxide)	Cold Sterilization
Rubber Polishers					
Classic Rubber Polishers	∅	√	√	√	√
Silicone Rubber Polishers	∅	√	√	√	√
Polyurethane Rubber Polishers	∅	√	√	√	√
Hi-glaze Diamond Porcelain Polishers	∅	√	√	√	√
Hi-gloss Diamond Composite Polishers	∅	√	√	√	√
Carbide Burs (Goldies & Uncoated)					
FG Composite Trimming & Finishing	√	√	√	√	√
FG Metal Cutting	√	√	√	√	√
Euro Carbide Goldies	√	√	√	√	√
Euro Carbide Burs	√	√	√	√	√
Carbide Bur Goldies	√	√	√	√	√
Carbide Burs	√	√	√	√	√
Stones					
Green Giant Stones	√	√	√	√	√
White Arkansas Stones	√	√	√	√	√
A/O Rubi Stones	√	√	√	√	√
Diamonds (Goldies & Uncoated)					
Diamond Points & Discs	√	√	√	√	√
Sintered Slims - Solid Diamond Discs	√	√	√	√	√
Dia-Dome - Diamond Sanding Caps	√	√	√	√	√

√ = Recommended

∅ = Not Recommended

Do Not Use ultrasonic cleaner for stones and rubber polishers